Sampling Form MRD 2

This form is essential for transport of the samples without any delay. It is commonly known as the shipping document or the (air)way bill.

Commonly this document is specific for each courier. Currently many of the courier companies will have website based Forms that are printed by the courier.

During transport, the product is accompanied with this Form 2 stating:

- 1. Date (and time) of sending
- 2. Collection address of the center that sends the package, with the data of the contact person
- 3. Receiver/Delivery address of the recipient referral laboratory, with the data of the contact person

The samples are **<u>not</u>** classified as dangerous goods.

Make sure to write in the description field: Biological Substance, Category B, UN3373

General Description Plass pet full details on commercial involta	Number of Nerro	Weight		Dimensions		
		KAn	Grame	Length	Wah	Height
		-	1000			0
		-				
Not No. Total		-	-	Competent adaptit to observe in Pass who to us bothors a and formers bothors a		

The weight is commonly 100 grams and the dimensions: Length 20, Width 10 and Height 3.

It is crucial for the quality of the sample that the package is kept on room temperature. This can be filled in as "Ambient" in the *Special Delivery Instructions*.



It is best to prepare the package for pick up before 15.00 hr, which will largely reduce the costs of the express sending.